

NEW

N-Histofine® ALK Control Slides

Each slide contains two formalin fixed paraffin embedded cell lines, NCI-H2228 (+: Positive) and SK-BR-3 (-: Negative).


Code: 417081F Package: 5 slides

- Use this product with N-Histofine® ALK Detection Kit (Code No. 417071F).
- This product is used as a standard for sorting out the positives tissue blocks which have expressed ALK protein from many tissue blocks.
- This product is used for validation of reagent performance and staining technique in the IHC-staining (including cytology staining) with N-Histofine® ALK Detection Kit.
- Read the package insert of N-Histofine® ALK Detection Kit carefully before using this product.

Staining results


Cell line	Staining pattern	ALK Protein
NCI-H2228	Strong dark brown stains are observed in the cytoplasm of the majority of cells. Note: Since all cells are not always stained, the stained cell should be recognized to be positive findings.	Positive
SK-BR-3	No stain is observed in the cytoplasm of all cells.	Negative

■ Negative control cell line/SK-BR-3

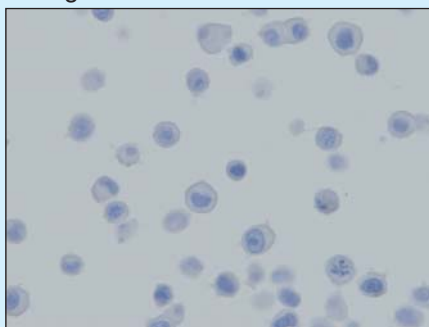

No stain is observed in the cytoplasm of all cells.
Treatment with hot bath (+)

■ Positive control cell line/NCI-H2228

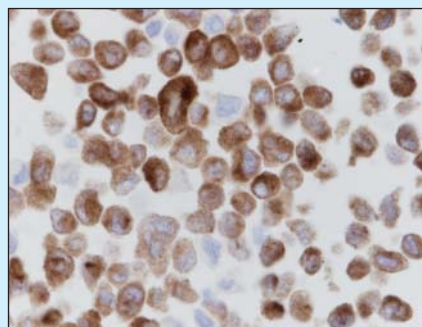

Strong stains are observed in the cytoplasm of the majority of cells.
Treatment with hot bath (+)

Used kit: N-Histofine® ALK Detection KIT (Code No. 417071F)

Manufacturer

NICHIREI BIOSCIENCES INC.

6-19-20, TSUKIJI, CHUO-KU, TOKYO 104-8402, JAPAN

PHONE: 81-3-3248-2207

FAX: 81-3-3248-2243

<http://www.nichirei.co.jp/bio/english>